

— Soft laser expertise

premio 30

Laser duo



Acupuncture
Ear Acupuncture

Sedatelec


premi^o 30

Laser duo

Painless stimulation of body and auricular points



**4 treatment
modes**

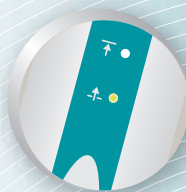

**2 powers,
adaptable for
superficial or
deeper points**



**10 hours
working time**
Rechargeable
battery

Between tradition and modernity: **safe and effective**

Acupuncture

Tonification

Controlled, rapid
tonification
of acupuncture points.



Dispersion

In a few seconds,
the Qi will be disperse
and you will remove
energy stasis.



Harmonisation

This treatment **rebalances**
the point and the circulation
in the meridian
by alternating impulse rates.



Ear Acupuncture

Frequency stimulation

This selection
will guarantee stimulation
of **auricular** points.




Innovation and effectiveness of frequencies

New impulse emission dynamics

Wobulation (100-10 000Hz)


Exponential acceleration

Tonification occurs as a result of an **acceleration** in **impulse emission rate**, which has an energising effect.

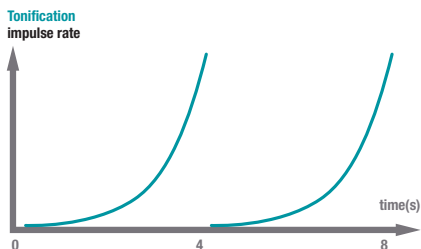


Exponential deceleration

Dispersion is obtained by **slowing** the **impulse emission rate**.

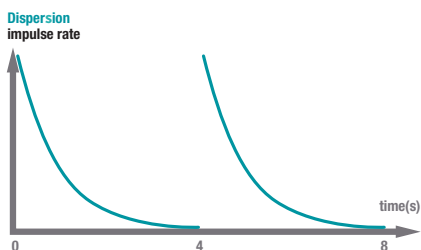

Alternating acceleration and deceleration

Alternating **acceleration** and **deceleration** of the emission rate leads to harmonisation of the point.

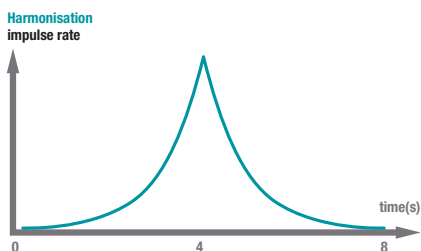

Scanning

According to the 7 Nogier* frequencies

The message is contained in the frequency for this ear acupuncture programme. It provides a successive emission by **scanning all the Nogier* frequencies** (from A to G, 73 - 4 672 Hz).


** The application of Nogier frequencies with this device has not been authorised by Health Canada for lack of sufficient clinical studies.*




Renowned experience in Pulsed Laser

Wavelength

The **905nm infra-red emission** passes through the skin barrier and reaches the deep cells, entering entirely safely without being absorbed by skin water, haemoglobin or melanin.

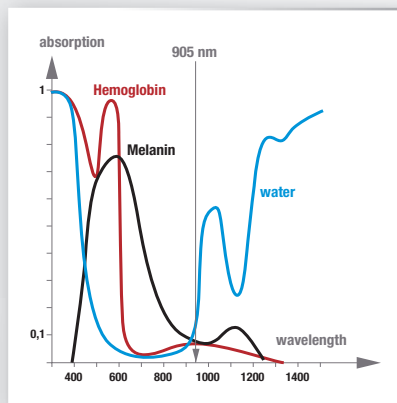

Peak value

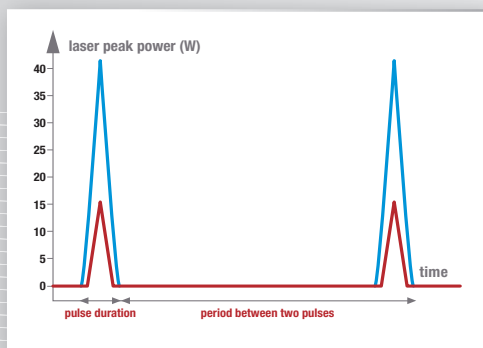

The laser beam is emitted as short impulses of high peak power. This method of emission gives the laser beam the **necessary power to enter tissues** entirely safely with no destructive heat effect, carrying the useful information to the site point.

Adjustable power

15 or 40 Watts power is adjustable depending on the desired depths

- **Superficial points:** surface points (face, ears)
- **Deep points:** other body points

Application time

Short treatment of 30 seconds for each point. This application time provides the optimal energy per point taking account of the physiological body reaction time.

Sedatelec +:

frequencies wobulation for acupuncture

The Sedatelec signature:

Nogier* frequencies for ear acupuncture



premio 30

Laser duo



**Effective, light
and ergonomic**

**A wireless laser,
ideal for the
consulting room
and on the move**



Invisible laser emission. Do not point the laser beam towards eyes. 3B classification according to IEC60825-1

Caution : Federal law restricts this device to sale by or on the order of a physician.

Your distributor

CE 0123

Made in France



ELECTRO-THERAPEUTIC DEVICES INC.

70 Esna Park Drive Unit 4,
Markham, Ontario Canada L3R 6E7
T:(905) 475-8344 | F:(905) 475-5143

MORE THAN JUST ACUPUNCTURE

